



Z Wave^{Pro}

Radial Pulse
Technology




Zimmer



Results of Z Wave^{Pro} Radial Pulse Technology

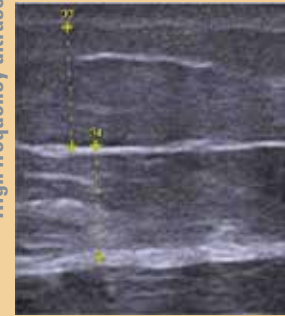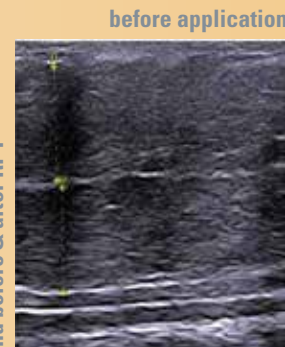

High frequency ultrasound before & after RPT

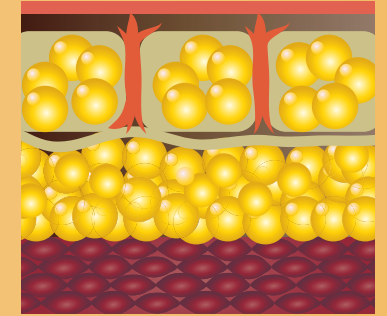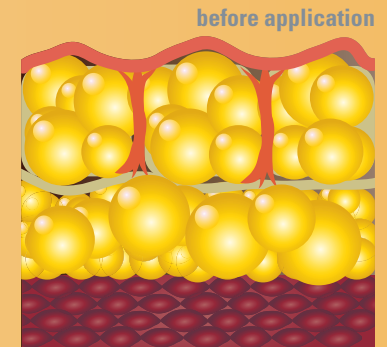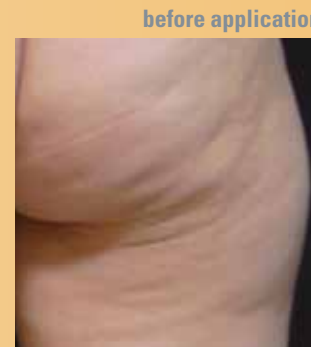

Z Wave^{Pro} – For the treatment of cellulite and to enhance body contouring results

| Indications | Cellulite I | Cellulite II | Cellulite III |
|--|--------------|--------------|---------------|
| Number of sessions | 8 – 10 | 9 – 12 | 10 – 14 |
| Applications per week | 2 – 3 | 2 – 3 | 2 – 3 |
| Frequency | 14 – 16 Hz | 10 – 16 Hz | 10 – 16 Hz |
| Power level (I / II / III) | 60 – 120 mJ | 90 – 185 mJ | 90 – 185 mJ |
| Radial pulses per application (per area) | 2500 to 3000 | 2500 to 3400 | 3200 to 3700 |



The difference between Z Wave^{Pro} vs. hand massage



Photo Courtesy of Dr. John Shieh, South Pasadena, CA

Advantages of Z Wave^{Pro}

- gentle, painless and quick treatments
- no disposables
- no down time
- non-invasive treatment
- easy to operate
- consistently reproducible results
- visible results



Body contouring

“Our 2016 Cryolipolysis® revenue was the best we had for the past few years. In spite of increasing Cryolipolysis® competition in the area, our business more than doubled from 2015 to 2016 since we added Z Wave. It was Z Wave that helped us stand out with a competitive advantage in our marketplace.”

Steven Weiner, MD
Facial Plastic Surgeon
Santa Rosa Beach, FL



Cellulite

“I was pleasantly surprised to see how many of our patients are enjoying its results as a stand-alone treatment for cellulite. It’s really the first cellulite reduction system that has delivered consistent results in our hands. More importantly, the patients are extremely satisfied and in addition to repeating treatments, they are telling their friends about it.”


Ross Kaplan, MD
Coastal Dermatology
Camarillo, CA



Cellulite Treatment with Z Wave^{Pro} Radial Pulse Technology

Hand piece characteristics

- efficient energy transfer from hand piece to deep tissue structures up to 35 mm
- treatment is comfortable for both patient and technician
- hand piece is controlled via foot switch
- each Z Wave^{Pro} comes with 2 hand pieces and is ready for use any time
- each hand piece is warranted for 2,000,000 pulses. The two hand pieces should yield approximately 1,600 application zones

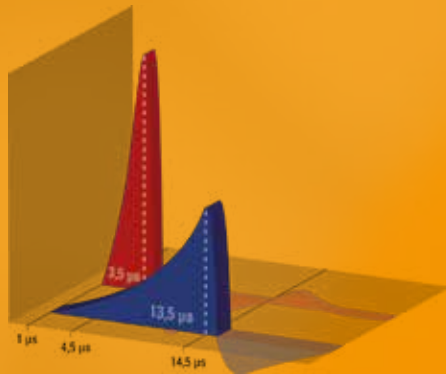

enhanced patient comfort due to longer rise time (pulse width) without the need for unpleasant high peak energy




Mode of action

The Zimmer Z Wave^{Pro} transmits powerful sound waves into the tissue. The energy then spreads out radially to enhance the treatment effect.

The body responds to the Z Wave^{Pro} sound waves with increased metabolic activity in the form of lymphatic drainage and increased blood circulation.

The end result is an improvement in the appearance of cellulite and other body contouring procedures.

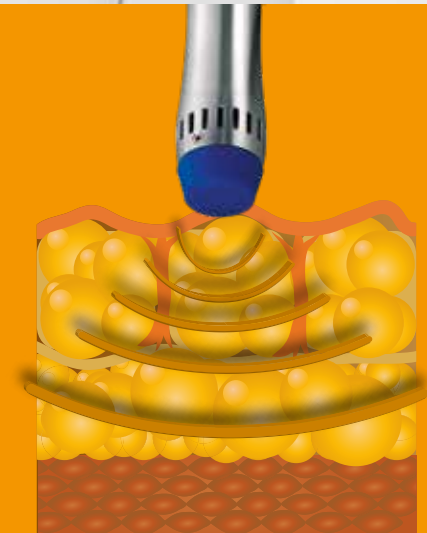



Product Characteristics

- simultaneous connection of 2 hand pieces
- intuitive and user-friendly touch screen operation
- energy levels from 60 to 185 mJ
- pulse frequencies from 1 to 22 Hz
- pulse counter
- menu with detailed application recommendations
- SD card for memory, updates and service
- proven reliability in the market place



Technical data



| | |
|-------------------|---|
| Technology | Electro-magnetically generated Radial Pulse Technology, 1-22 Hz, 60-185mJ |
| Programs | 3 recommended application presets |
| Control unit | 12" color touch screen for all software operations. Full range control for energy and frequency. Foot switch control for hand piece |
| Memory and Update | SD card for custom programs by user/error log/firmware updates |
| Voltage | 110 volts, 60 Hz |
| Dimensions | L 20.5" W 21" H 54.5" Weight 70 lbs. |
| Warranty | 2 years of limited warranty on system; 2,000,000 pulses on each hand piece |