

Z Wave^{Pro} Maintenance Guidelines



Zimmer

AestheticDivision
3 Goodyear Suite B
Irvine, CA 92618
Phone (800) 327-3576
info@zimmerusa.com
www.zimmerusa.com
shop.zimmerusa.com

Best Practices

RECOMMENDED LUBRICANT: Zimmer recommends that customers use either the Zimmer oil or mineral oil. Best practice is to place a few drops of oil on the skin. Then spread the oil using the hand piece in the off position and the silicone cap attached. With the hand piece already in position for treatment, simply press the foot pedal to begin treatment. Use only enough oil to allow the delivery system to glide freely over the treatment area. This reduces the risk of splatter and prevents oil from getting inside of the hand piece.

GEL LUBRICANT: Zimmer has found that some customers are using gel as a lubricant, either the gel used with the cryolipolysis[®] treatment or added ultrasound gel. As these gels are water based, they can corrode the inner mechanism if they get inside of the hand piece. Use of gel will void the manufacturer's warranty.

SCREEN SENSOR: If the screen becomes unresponsive to touch, it may be due to a film build-up. A mild window cleaner, like Windex[®], may restore the screen's touch sensitivity.

SILICONE CAP: After every treatment, Zimmer recommends that you pull the silicone cap off of the applicator head and clean all surfaces with alcohol. Ensure that the applicator head is screwed on tight to avoid getting oil in the pulse generator. For best cap longevity, it is recommended that the caps are cleaned with alcohol rather than a stronger disinfectant.



Zimmer

The Treatment is Radial Pulse Therapy (RPT) - The Device is Z Wave^{Pro}

MECHANICAL DEVICE: The sound from your hand pieces can vary. If you sense a system is delivering low energy, the best test is to compare how the hand pieces feel when delivering energy to tissue.

WARRANTY WARNING: A warning will appear on the screen as a hand piece approaches 2 million pulses, meaning that it will be out of warranty and no longer subject to a free replacement.

PULSE COUNT: Each delivery system's pulse count can be found by selecting "Settings" on the Home Screen. On the left, "Hand Piece Counter Status" shows how many pulses are on each delivery system. The #1 hand piece is to the left; #2 is on the right. Use this counter to track when to replace tips. Record the hand piece pulse count at the time when you put on a new tip to track that tip's use count.

TWO HAND PIECE OPERATION: When treating more than one area in a session, Zimmer recommends you alternate hand pieces each time you move to a new treatment area. To change to the alternative delivery system, press the screen icon in the upper right. If necessary, refer to the User Manual for instructions to switch from Hand Piece 1 to Hand Piece 2.

CLEANING THE APPLICATOR HEAD: Remove the silicone cap. Unscrew the applicator head. Clean the inside of the applicator head with an alcohol swab. Clean the area inside the hand piece with an alcohol moistened Q-tip®.

CHECKING THE APPLICATOR HEAD: We suggest that the applicator heads be checked regularly for wear beginning at 250,000 pulses. If you sense less energy is being delivered or that the sound is diminished, first inspect the applicator head for wear and possible replacement. Tips should be replaced at or before reaching 500,000 pulses.

HAND PIECE AND HAND PIECE HOLDER: After each use, wipe clean and disinfect with alcohol.

