

ZShape

New generation
of bodyshaping



Zimmer
MedizinSystems

Z SHAPE: COMBINING VACUUM AND RF FOR CELLULITE REDUCTION BODY CONTOURING

Our new generation

VACUUM

The medium and large handpieces incorporate a vacuum function that enhances blood circulation, and stimulates metabolism.

RADIO FREQUENCY (RF)

Surface and vacuum chamber electrodes ensure uniform and deep delivery of RF energy across the entire treatment area.

The synergy of RF and Vacuum technologies boosts treatment success and patient satisfaction.



HANDPIECES

**PERFORM RADIOFREQUENCY
TREATMENT LIKE NO OTHER**

MULTIFUNCTIONAL HANDPIECES

Z Shape's unique handpieces have each been specifically designed to accommodate a range of treatment areas.



small



medium



large



INTELLIGENT FEATURES FOR EFFORTLESS AND COMFORTABLE TREATMENT SESSIONS

EACH HANDPIECE PROVIDES REAL-TIME FEEDBACK ON THERAPY:



COUPLING INDICATOR:

Solid LED handpiece indicator begins flashing if optimal skin contact is lost or if gel is required.



TEMPERATURE INDICATOR:

Real-time temperature feedback assures safe and therapeutic temperatures.

blue LED = sub-optimal temperatures

green LED = optimal treatment temperatures

orange LED = excessive temperatures

In addition, the precise temperature is continuously displayed on-screen.



CONTROL AT YOUR FINGERTIPS:

Seamlessly control the therapy by adjusting RF energy intensity directly from the handpiece, eliminating the need to adjust settings on the device screen.

Patient & Practitioner Benefits



PAIN-FREE

Z Shape is a very comfortable treatment, even for people sensitive to pain. Patients feel a pleasant warm sensation and a gentle massaging effect from the vacuum.



NON-INVASIVE

Unlike many other treatments, *Z Shape* is a completely non-invasive. No needles, just a handpiece gently gliding over the skin.



NO DOWNTIME

Patients can immediately return to work or attend their evening event directly after the treatment.



MINIMAL TIME COMMITMENT

Prior to therapy, a brief skin cleansing is all that's needed. No post-treatment care is required, reducing treatment time for both staff and patients.



ZShape

USER INTERFACE

- » Intuitive design
- » Practical presets for different body regions
- » Individual adjustment options for maximum efficiency
- » Can save "Favorites" settings

Easy-to-use controls, either by touchscreen or rotary knob. Just tap to select your treatment area, and the ideal handpiece and settings are automatically pre-selected for you. You can also make personal adjustments anytime and save them as "Favorites."

PERSONALIZE YOUR TREATMENT

- » Z Shape offers three effective treatment frequencies.
- » Choose between constant vacuum or one of three pulse modes.
- » Selectable vacuum levels for optimal outcomes.



ADVANCED RADIOFREQUENCY AND VACUUM TECHNOLOGY FOR TEMPORARY REDUCTION OF CELLULITE AND SKIN TEXTURE

TO PROVIDE DEEP TISSUE HEATING
AND TO INCREASE BLOOD FLOW
IN THE TREATMENT AREA



Small handpiece



Medium handpiece



Large handpiece

DÉCOLLETÉ
NECK



ARMS

ABDOMEN

BUTTOCKS

THIGHS

CALVES



TECHNICAL DATA

LCD touchscreen

3 handpieces

4 different pulse modes

0–10 vacuum level

1 MHz, 2 MHz, 4 MHz output frequency

1–20 RF intensity levels

46.5^H 22.4^W 22.1^L Dimensions (in)

66^{lb} Weight